

UbiDuo

The UbiDuo allows deaf and hard of hearing people to communicate with hearing people in a natural way. It consists of two keyboards with very small screens. When one user types something, it appears in real time on the other user's screen.

Using the UbiDuo:

1. **Open the UbiDuo like a book and lay it on a flat surface.**
2. **Separate the halves by pulling the top hinge up and the bottom hinge down at the same time.**
3. **Turn on each half of the UbiDuo by pressing the little green power button on right hand side. Hold the button down until the screen lights up.**
4. **Press Enter to choose Split. This will take you to the split screen.**
5. **Start typing. Each letter appears as it is typed, so do not worry about spelling or grammar -- so long as your conversation partner understands what you're trying to communicate, you're doing fine.**

Tip: If you have difficulty reading the screen, adjust the angle.

When you are finished with the UbiDuo:

1. **Turn each half off by pressing the little green power button. Hold the button down until the screen goes dark.**
2. **Attach the two halves by lining the hinges up with the holes and pressing them in at the same time.**

To Change the font size on the UbiDuo:

1. **Press Esc to return to the menu screen.**
2. **Use the arrow keys to select Settings and press Enter.**
3. **Use the arrow keys to select Font and press Enter.**
4. **Use the arrow keys to select the desired font size and press Enter.**
5. **To return to the split screen, press Esc to go back to the main menu, then use the arrow keys to select Split and press Enter.**

If your conversation partner has their own UbiDuo, use the arrow keys to choose Auto-Link from the menu screen and press Enter. The UbiDuo will find any other UbiDuo within its range and connect to it.